



FACE MASK


Disposable, Non-Medical Adult Mask

P518025

QTY: 50 PIECES


PRODUCT DESCRIPTION

- Intended for general purpose
- Single-use mask
- Adult size – one size fits most
- 3-layer protection
- Flexible nose bridge
- Elastic ear loops, latex-free
- Comfortable and breathable
- Comprised of polypropylene, polyester and spandex
- Color: white
- Country of origin: USA

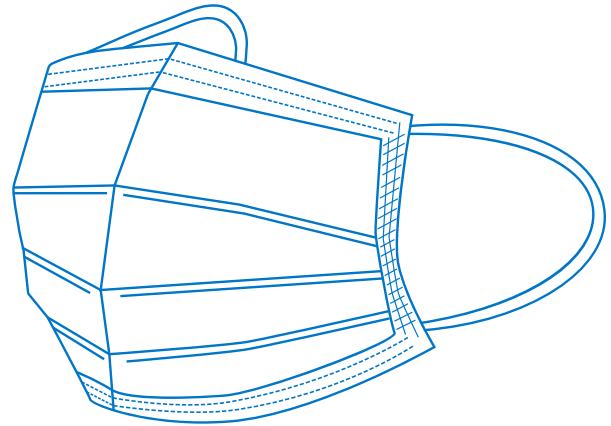

INSTRUCTIONS FOR USE

Failure to follow instructions on use of this mask will reduce the effectiveness of the mask. Do not use if damaged or soiled.

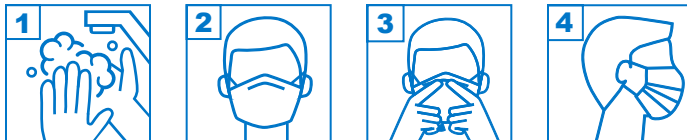

1. Wash hands thoroughly.
2. Place mask over nose and mouth. Place ear loops around each ear.
3. Adjust nose piece to fit tightly across nose.
4. Fit mask underneath the chin.

Dispose of face mask properly after use.

WARNING

- Not approved for use in a medical/surgical setting or where significant exposure to bodily fluids, other hazardous fluids, high-intensity heat sources, or flammable gas is expected.
- Not approved for use in clinical settings where risk of infection through inhalation exposure is high.
- Take care when wearing mask with glasses as glasses may fog—especially when driving or operating machinery. Use anti-fog glass wipes to reduce fogging.
- This mask has not been FDA cleared or approved, as it is a non-medical mask.
- This mask has been authorized by FDA under an EUA for use as source control by the general public to help prevent the spread of infection or illness during the COVID-19 pandemic.
- This mask is authorized only for the duration of the declaration that circumstances exist justifying the authorization of the emergency use of medical devices, including alternative products used as medical devices, during the COVID-19 outbreak, under section 564(b)(1) of the Act, 21 U.S.C. § 360bbb-3(b)(1) unless the authorization is terminated or revoked sooner.